



COLO QUICK



GUIDE FOR SILICONE
APPLICATION ONTO
BEARING COMPLETE

Table of contents

Procedure explanation and safety.....2

Tools needed.....2


Procedure walkthrough.....3

Procedure explanation and safety

This guide explains the procedure of applying silicone onto the bearing complete.


- Children and animals must never be left alone with the coloQuick. Be especially careful when the top lid and/or service door is open.
- There is a risk of electric shock when the service door is open. It should only be opened by authorized personnel trained by Calvex A/S.
- This operation must only be performed by authorized personnel trained by Calvex A/S.

Tools needed

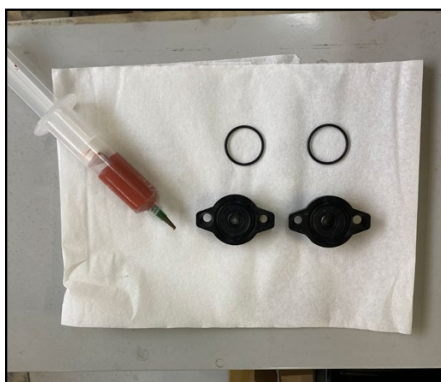


Tube with silicone (605010),
two sets of Bearing complete
(803101)


Procedure walkthrough

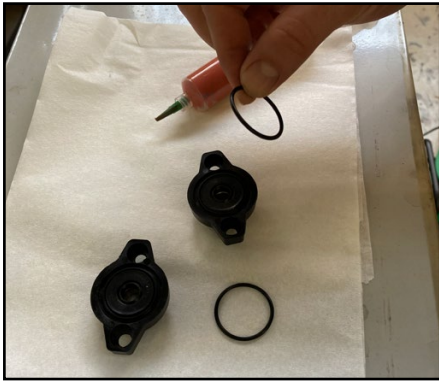

Place the O-ring at the bearing.

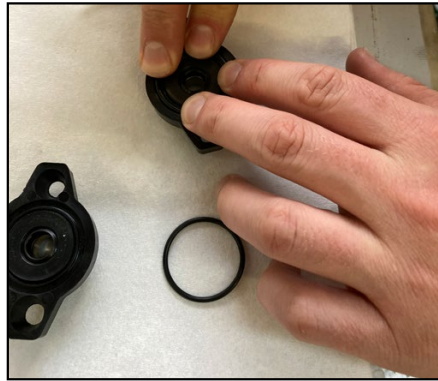

Use fingers to press it into the groove.

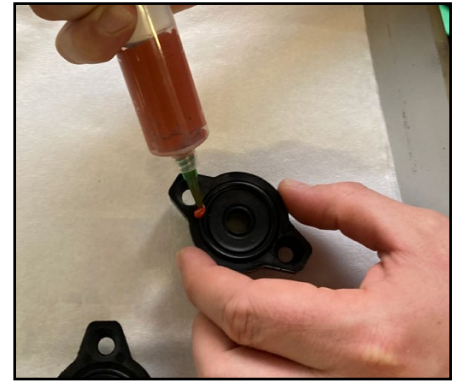

Apply the silicone **ONLY** on the outer side of the O-ring.



Distribute the silicone evenly all around.



Repeat on the second bearing.



Make sure the silicone is **ONLY** applied in the area marked with a green circle on the above image.



It is advised to let the silicone dry until the next day before filling the waterbath and heating the water. This ensures that the bearing does not leak. Silicone dries 2 mm. in 24 hour.

