

## coloQuick Pasteur + MAXX4

# HOW TO UPDATE THE SOFTWARE

**1. Move the file called “Pasteur.hex” to an empty USB flash drive**

**2. Insert the USB flash drive**

- Pasteur: Left side of the front lid
- MAXX4: Just below the main switch

**3. Turn off the machine by using the red main switch**

- Wait for the machine to turn off



**4. Hold down the “PRG” button while turning the machine back on.**



**5. Wait for this screen to appear:**

- The software is now being updated

```
New application
Transport init.
App img. file found_
```

**6. The update is completed after 15-30 seconds**

```
SW Upgraded
```

**7. This screen will appear after a few seconds, showing the updated software version.**

- In this example, version 2.47a.

```
Pasteur Maxx 4
ver. 2.47a / 251
BSL: 5
```

The coloQuick unit is now ready for use, and the USB flash drive can be removed.

